**TOPRO
TROJA**

TOPRO

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Welcome as a user of a TOPRO PRODUCT

Every TOPRO product is developed, designed and manufactured in Norway. Due to advanced manufacturing by using the best equipment we are able to offer great accuracy and maximum quality in all levels of our production. This product conforms to the Medical Device Directive 93/42/EEC.

It is tested and approved according to ISO 11199-3:2005. On receipt of the goods please check your product, if you have any queries please contact your mobility shop or dealer immediately.

Guarantee

TOPRO TROJA WALKER is guaranteed free of faults and defects for a period of 5 years.

An exception is if the damage is caused by incorrect use or parts that are exposed to natural wear and tear (such as e.g. brake block, brake cable, wheel, seat and handgrip). For repairs during the guarantee period, please contact your local mobility shop or dealer. The guarantee is void if unauthorised spare parts and accessories have been or are being used on the product.

The red numbered squares below refer to the numbered illustrations on the inside of the cover.

1 Get to know your TOPRO PRODUCT

- | | | |
|--|-----------------------------|------------------------------|
| A Forearm support | F Seat bars | K Basket |
| B Brake handle | G Seat | L Side frame |
| C Brake lever | H Lock clamp | M Brake block / Edge guard |
| D Forearm support tube | I Strap for folding | N Tilt function |
| E Brake wire | J Lock wheel for the handle | O Supporting cord |
| P The label on the rear side of the tube under the seat shows the product name/model and | | |
| • Maximum weight of user | • Maximum width of rollator | • Maximum height of rollator |

Measurements

Model/size	TOPRO TROJA M		TOPRO TROJA S		TOPRO TROJA X	
	Dark grey		Silver grey		Silver grey	
Colour	CM	Inches	CM	Inches	CM	Inches
Recommended user height	150 - 185	4ft 11" – 6ft 1"	130 - 170	5ft 1¼" – 5ft 7"	130 - 160	5ft 1¼" – 5ft 3"
Height of support points	98-115	38 ⁵ / ₈ " - 45¼"	84-100	33" – 39 ³ / ₈ "	84-96	33" - 37¼"
Maximum length	65	25 ⁵ / ₈ "	65	25 ⁵ / ₈ "	65	25 ⁵ / ₈ "
Maximum width	61	24	61	24	52,5	20½"
Between support points	45	17¾"	45	17¾"	35	13¾"
Seat height	62	24 ³ / ₈ "	54	21¼"	52	20½"
Turning diameter	84	33 ¹ / ₈ "	84	33 ¹ / ₈ "	84	33 ¹ / ₈ "
Diameter wheels	20	7 ¹ / ₈ "	20	7 ¹ / ₈ "	20	7 ¹ / ₈ "
Width wheels	3,6	1 ³ / ₈ "	3,6	1 ³ / ₈ "	3,6	1 ³ / ₈ "
Grip distance driving brake	7,4	2 ⁷ / ₈ "	7,4	2 ⁷ / ₈ "	7,4	2 ⁷ / ₈ "
Height when folded	108	42½"	88	34 ⁵ / ₈ "	88	34 ⁵ / ₈ "
Width when folded	31	12¼"	31	12¼"	31	12¼"
Length when folded	66	26	66	26	66	26
	KG	LBS	KG	LBS	KG	LBS
Weight walker	10,1	22lbs	9,8	21.8lbs	9,3	20.6lbs
Maximum weight of user	135	298 (21st 3.6lb)	125	274 (19 ½ st)	80	176 (12½ st)

Intended use

The TOPRO Troja Walker is designed and approved to be used indoors on plain foundation. The walker safely supports users with poor balance and/or poor mobility.

Important to know about safety before use

- Take care when folding and unfolding, do not trap your fingers.
- Check the brakes every time before use, and that the rollator locks into position when opened.
- Not to be used by persons over maximum user weight (see table).
- The basket has a max load of 5 kg.

- Do not use to transport heavy loads or persons.
- The walker is not a toy.
- Make sure that the lock wheel for height adjustment for the handles is locked tight.
- Be aware! Some parts of the rollator might feel hot or cold when exposed to extreme temperatures.

2 Important to know about safety when in use

- When stationary or parked, always use the parking brakes on both sides **A**
- Before sitting on the seat, make sure the parking brakes on both sides are in the locked position **A**
- Do not lean or bend sideways or over the back of the rollator whilst in the seated position.
- The driving brakes can be held in a little **B** to reduce speed, but must not be in locked position when the walker is in motion.
- To give the right support and a safety, make sure the walker is kept as close to your body as possible **C**
- Walk straight, look forward and hold your backside in **D**. That is the easiest way to keep your balance.
- Do not push the walker too far from your body as this can cause unnecessary strain and you could fall.
- Be careful of flared skirts and trousers as they could get caught between the brake block and wheel, which could lead to falling. We therefore recommend not wearing flared/wide clothing.

Assembling / adjusting

TOPRO TROJA is delivered assembled. Before use, you must unfold the rollator. The height of the handlebar tubes will then need adjusting to suit the users needs.

3 Unfolding / folding

Unfolding: Loosen the strap between the forearm supports **A**. Loosen the lock clamp that is over the seat bars **B**. Press the seat bars down and out **C** The side frames will slide apart. Press down the seat bars. You will hear a clicking sound. The rollator is now locked in its open position. Fasten the strap **A**.

Folding: Pull the strap on the seat **D** and press the side frames together. Lock the clamp **E** over the seat bars.

4 Adjusting the handles and height adjustment of the forearm supports

Adjusting the handles: Unscrew the lock wheel **A** until the handle can be moved, adjust to the wanted position and tighten the lock wheel again. It is done separately on each side. It is not possible to pull the handles all the way out; there are physical stops both ways. Height adjustment: Unscrew the lock wheel **B** (approx. 4-5 turns). Adjust the forearm support tubes to a suitable height, then pull them slightly up and down **C** until you hear a clicking sound. Tighten the lock **B** again. Adjust both handlebars to the same height. **Recommended height: Stand with relaxed shoulders with your elbows in a 90 degree angle, see illustration 2.**

5 Height adjustment using the memory function

You have the opportunity to set the height of the forearm supports at a standard setting fitting to your height so that you easily can find the same height again. This can be necessary when the walker is being transported or stored as compact as possible. Decide the height of the forearm support tube. Count visible holes at the back of the tubes. Unscrew the lock wheel **A**. Pull the tube **B** out of the frame. The attached part **C** will now follow the tube. Move the attached part **C** on the tube **B** so there are equal holes as counted earlier. Fit the memory chip **D** on the tube in the first available hole under the attached part **C**. Push the tube **B** down and into the side frame **E** until the attached part is locked. Screw the lock wheel **A** half in. Pull the tube **B** up until it stops, and then tighten the lock wheel **A** properly. From now on, when the forearm support tubes are in the lowest position, they will stop in the preset position automatically when pulled up.

6 Adjusting / checking and maintaining the brakes

Adjusting / checking: Loosen the nut **A** beside the brake. Tighten the brake wire **B** with the adjustment screw **C**. Adjust both brake blocks so the distance to the wheel surface is approximately 1 mm **D**. Make sure the brakes are not adjusted too tight and pressing into the rubber when the parking brake is on.

Maintaining: Clean the brake block **E** and the brake spring **F** frequently. The wheels can be changed if they are worn out or damaged.

2 Using the brakes

Make sure the brakes work before each trip. Each brake handle **A** works on each of the rear wheels.

The rollator wheels are made of a special soft rubber which gives an optimal braking capacity on slippery floor. Wrong use can cause unsafe situations and damage to the walker: **The walker must not be driven or pushed with force when the parking brakes are on or when the wheels are partly locked using the brakes. This indicates incorrect use and can cause wear and tear which makes the tires and brake quality weaker.**

Parking brakes: Push both brake levers **A** forwards until they lock. Both rear wheels are now locked. Pull the brake levers back to release the parking brakes. *Driving brakes:* Squeeze both brake handles **B** upwards to reduce speed. **Do not use continuously while the walker is in motion.**

7 Seat and basket

The parking brakes must be on before using the seat **A**. Push both brake levers **B** forwards until they lock. Both rear wheels are now locked. Sit with your back towards the basket **C**. The basket **D** can carry goods up to 5 kg. It can be removed and washed separately and gently (handwash 40°C).

8 Edge guard and tilt function

Edge guard: The brake blocks **A** which are in front of the rear wheels prevent the rear wheels from catching on corners, door frames etc. Tilt function: To use the tilt function use the front of the foot and press down on the tilt pedal **B** which is beside the rear wheel. This helps to tilt the rollator over thresholds, edges etc.

9 Carrying the walker

The walker must not be carried by the seat strap or the brake wires. Fold the rollator together as shown on the illustration. Carry the walker by the folded seat bars **A**.

Material / Recycling

The rollator is made of plastic coated aluminium pipes, plastic couplings and material of plastic and polyester. Most parts can be recycled.

Cleaning

The rollator can be steamed, hosed and washed with normal household detergents.

Disinfection must only be carried out by trained personnel

The surface must be cleaned with a disinfection that contains 70-80% ethanol. We advise against disinfection that contains chlorine and phenol. *The manufacturer can not be held responsible for any damage or wounds that might occur by the use of harmful disinfection or disinfection carried out by unauthorized personnel.*

Storage

The rollator should be stored in an upright position. It can be stored in and outside, if stored outside then under cover.

Inspection / Maintenance / Re-use

It is recommended that general inspection and maintenance is carried out and that the items below are checked on a regular basis: Tubes and frame, handlebars and handlebar tubes, brakes, wheels, seat and accessories. This also applies when the rollator is to be prepared for re-use. Please see detailed instructions in this user manual about maintenance of the brakes and cleaning and disinfection of the rollator.

If the walker does not work properly

Make sure the rollator is unfolded correctly and that both brake handles are in expected position.

Accessories

For pictures and TOPRO article numbers see the inside of the cover. Accessories mounted on the rollator may influence stability, we therefore advise you to use them carefully.

A Tray	H Bottle holder
B Anti slip mat for tray	I Nameplate
C Crutch holder	J Lock
D Back support	K Flower stickers, set with 4 stickers
E Shopping bag	L Guiding handle for e.g. visually impaired
F Rear bag with zipper	M Drag brake (pair)
G Basket for oxygen bottle (only compressed, not liquid)	N Stabilizer bar

The accessories are to be ordered separately to give the possibility to compose a TOPRO TROJA WALKER that suits the individual users need. Contact your mobility shop, dealer or TOPRO for an updated overview of the accessories, or visit our homepage www.topro.no

In case of faults or questions please contact your mobility shop or dealer.

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